


This informational sheet is intended to help you determine if your laser imager is ready to print with DRYVIEW DVE Laser Imaging Film.

How can you tell if your laser imager supports DVE Film? See detailed instructions below.

1. Check the serial plate on the back of the laser imager.
2. Check your imager’s software version in the Web Portal or on the imager’s touch screen.

Check the Serial Plate

The serial plate is on the **back panel of the laser imager**.
Each imager’s serial number (SN) is shown after **SN** on the serial plate.

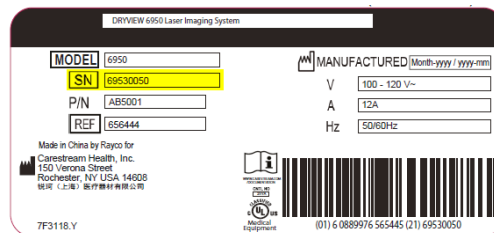

Serial Number Starting Points for DVE-capable Laser Imagers

Imager Models 5700, 5950, 6950 with the SN shown in the table (or higher) can use DVE Film.

- If the SN on the serial plate matches or is **higher** than the number in the following table, use DRYVIEW DVE Film.
- If the SN on the serial plate is **lower** than the number in the table, OR if your imager is model 5800, 5850, 6800, 6850, or 8900 continue with the next section (Check Imager’s Software).
- DRYVIEW models 8700, 8150 and 8300 are not compatible with DVE Film

Imager Model	SN Start for DVE Film
5700	57035971

Imager Model	SN Start for DVE Film
5950	59534777

Imager Model	SN Start for DVE Film
6950	69530368

Check if your Imager’s Software is DVE-capable through the Web Portal or the Imager’s Touch Screen

The imager can be updated to a software version that supports DRYVIEW DVE Film. Check the imager’s software version:

Imager Models 5700, 5950, 6950, 5800, 5850, 6800, 6850, 8900	Imager Models 6950, 6800, 6850, 8900
<p>Access the Web Portal:</p> <ol style="list-style-type: none"> 1. From a personal computer on the same network as the laser imager, start a WINDOWS INTERNET EXPLORER (IE) browser. 2. In the browser’s address field, type <a href="http://<imagerIPaddress>">http://<imagerIPaddress> and press Enter. Obtain the imager IP address from your IT group if not known. The Web Portal opens to the Device Status (Home) screen. NOTE: If using IE other than 6, 7, or 8, or if a “System Setup Message” appears, enable Compatibility View in the browser window. Locate Compatibility View from the Tools (Gear) menu. 3. On the left panel, select System Information > Version Information. Note: The screens differ slightly between imager 	<p>Check the software version on the imager’s touch screen.</p> <p>For 6950, 6800, 6850: Touch Utilities > View System Info > System Info > Versions.</p> <p>For 8900: Touch Menu > System Information > Version</p> <p>The software version displays on the screen.</p>

View the software version:

- If the software version matches or is **higher** than the version in the table below, the imager supports use of DRYVIEW DVE Film. Contact Carestream sales or your Carestream film dealer to order DRYVIEW DVE Film.
- If the software version is **lower**, contact Carestream service or your authorized Carestream service partner for a software update

Imager Model	Minimum Software
5700	1.8.b2
5950	1.5.b2

Imager Model	Minimum Software
6950	1.3.b2
5800/5850	2.7.b1

Imager Model	Minimum Software
6800	2.08.b1
6850	1.09.b2

Imager Model	Minimum Software
8900	V3.80.b1

Note: Please do not order DVE film until your software has been updated or your laser imager is ready.